



Dental Materials, Inc.

ZR-P Primer

REF 90577, 90578
Rx ONLY

Description:

ZR-P Primer is a single component priming agent used to enhance adhesion when applied to zirconia, alumina and metal/alloy surfaces. ZR-P will significantly increase bond strength between a resin-based material and the bonding surface of the restoration. ZR-P is offered in both a syringe and bottle.

Indications for use:

ZR-P is intended for use as a surface treatment for the following restorations

- Zirconia
- Alumina
- Metal/alloy
- Composite
- Endodontic Post
- Intraoral Repairs

Instructions for use:

1. (Optional*) Microetch the intaglio surface of the restoration, rinse and air dry
2. Cleanse the restoration with an appropriate cleanser
3. Apply 2 coats of ZR-P to the prepared surface, air dry thoroughly
4. Continue protocol with adhesive and / or cement of your choice according to the manufacturer's instructions

(*) Microetching or roughening is not required, however it does create a more ideal surface for bonding

For Intraoral Repairs or Zirconia/Alumina or Metal Restorations

1. Isolate the repair area
2. Roughen (*) the surface of the restoration using a diamond bur or air abrasion. Rinse and dry thoroughly.
3. Cleanse the restoration with an appropriate cleanser
4. Apply 1-2 coats of ZR-P. Air thin/dry for 3-5 seconds
5. Continue protocol with adhesive of your choice according to the manufacturer's instructions

(*) Microetching or roughening is not required, however it does create a more ideal surface for bonding.

NOTE: Contamination to the internal surface of a restoration will cause a decrease in bond strength. If delivering zirconia, do not cleanse the surface with phosphoric acid as it can make the surface less receptive to bonding.



Dental Materials, Inc.

ZR-P Primer

Precautions and Warnings:

1. If using from a syringe to avoid clogging, remove tip and recap with syringe cap. Verify material flow prior to applying intraorally.
2. Avoid contact with eyes, skin and soft tissue. If skin is exposed to ZR-P, wipe area with alcohol and thoroughly rinse with water. In case of contact with eyes, rinse immediately and seek medical attention
3. If allergic reaction develops, discontinue use and consult doctor
4. When not in use, store in refrigerator at 2C-8C (36F-46F)
5. Patient, clinician and assistant are recommended to wear protective eyewear when using ZR-P
6. Do not use after expiration date
7. To avoid cross contamination, re-cap, and wipe syringe with a disinfectant between uses
8. Dispose of used tips and empty syringes
9. Keep product out of direct sunlight/heat
10. Keep out of reach of children
11. For professional use only

NOTE: For additional information or SDS sheets please visit apexdentalmaterials.com.



WARNING

Causes serious eye irritation. Wear eye/face protection. Prolonged or repeated skin contact may cause severe irritation.



FIRST AID: IF IN EYES: Rinse cautiously with water for several minutes, also under eye lids. Remove contact lenses, if present and easy to do so. IF INHALED: Move to fresh air and keep at a rest position comfortable for breathing. IF INGESTED: Do not induce vomiting. Rinse mouth, drink plenty of water. Call a physician or Poison Control Center immediately.



Inter-Med, Inc.
2200 South St.,
Racine, WI 53404
Tel: +1-262-636-9755

